

RAVEN® Litter Series

Raven® II Model 90C Litter: PN:60-0001 NSN:6530-01-432-5114 CE

INSTRUCTIONS FOR USE

- **STEP 1:**
While standing upright, release buckles from litter.



- **STEP 2:**
Place litter on the ground and completely extend with the fabric side up.

NOTE:
When litter is in the relaxed position, the hinges closest to you (when holding the handles) will look like Figure 1.



Figure 1



NORTH AMERICAN RESCUE®

www.NARescue.com • Greer, SC 29650 • 888.689.6277

PRODUCTS WITH A MISSION®


RAVEN® Litter Series

Raven® II Model 90C Litter: PN:60-0001 NSN:6530-01-432-5114 CE

(OPENING INSTRUCTIONS CONTINUED)

- **STEP 3:** Keeping the litter straight as possible, grab handles and rotate inward until all hinges rotate & lock (see Figure 2).

NOTE:
This action is best executed simultaneously using 2 individuals on each end of the litter.

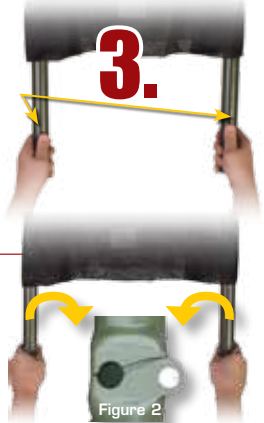


- **STEP 4:** While maintaining hinges in the locked position, apply firm, steady pressure on the spreader bar with foot. Increase pressure with foot until spreader bar locks into place.

NOTE:
To close and store litter, repeat the directions in reverse order.



- **STEP 5:** Place patient on litter, securing the straps around the patient, buckle and pull straps tight to avoid patient movement during transport.

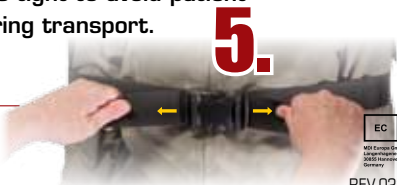

EC REP
M3 Corp GmbH
Lapphäger Str. 71
3852 Haverlar-Lagebagen
Germany

REV.022310



NORTH AMERICAN RESCUE®

www.NARescue.com • Greer, SC 29650 • 888.689.6277

PRODUCTS WITH A MISSION®